

Coronavirus COVID-19

BC Centre for Disease Control | BC Ministry of Health



Cleaning and Disinfection Instructions for Eye/Facial Protection

Required Equipment

Cleaning products required:
Soap (e.g. dish soap) and
disinfectant wipes (e.g. Accel wipes)

2 PPE required:

Exam gloves (Vinyl or Nitrile)

A Reusa

Reusable Eye Protection (goggles, safety glasses, faceshields without foam)









If eye protection appears damaged or compromised: DO NOT REUSE.

If reusable eye protection is visibly contaminated or soiled:

- 1. Don new pair of exam gloves.
- 2. Using a clean cloth, wipe with soap and water, cleaning from the inside to the outside.
- 3. Rinse with water & remove excess water.
- Using one disinfectant wipe at a time, and first squeezing excess disinfectant into a sink to prevent splash to face, thoroughly wipe the interior followed by the exterior of the facial protection.
- Ensure all surfaces remain wet with disinfectant for at least 1 minute (or applicable disinfectant wipe contact time).
- Equipment may be rinsed with tap water if visibility is compromised by residual disinfectant.
- 7. Allow to dry (air dry or use clean absorbent towel).
- 8. Remove gloves and perform hand hygiene.

If reusable eye protection is not visibly contaminated or soiled:

- Don new pair of exam gloves.
- Using one disinfectant wipe at a time, and first squeezing excess disinfectant into a sink to prevent splash to face, thoroughly wipe the interior followed by the exterior of the facial protection.
- Ensure all surfaces remain wet with disinfectant for at least 1 minute (or applicable disinfectant wipe contact time).
- Equipment may be rinsed with tap water if visibility is compromised by residual disinfectant.
- 5. Allow to dry (air dry or use clean absorbent towel).
- 6. Remove gloves and perform hand hygiene.




Faceshield with Visor & Foam for Forehead Comfort





If foam piece is visibly soiled: DO NOT REUSE.

If faceshield appears damaged or compromised: DO NOT REUSE.

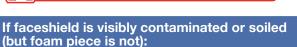

- 1. Don new pair of exam gloves.
- 2. Using a clean cloth, wipe with soap and water, cleaning from the inside to the outside.
- 3. Rinse with water & remove excess water.
- 4. Using one disinfectant wipe at a time, and first squeezing excess disinfectant into a sink to prevent splash to face, thoroughly wipe the interior, followed by the foam band, strap exterior, and exterior of the visor.
- 5. Ensure all surfaces remain wet with disinfectant for at least 1 minute (or applicable disinfectant wipe contact time).
- 6. Equipment may be rinsed with tap water if visibility is compromised by residual disinfectant.
- 7. Allow to dry (air dry or use clean absorbent towel).
- 8. Remove gloves and perform hand hygiene.


If faceshield is not visibly contaminated or soiled:

- 1. Don new pair of exam gloves.
- Using one disinfectant wipe at a time, and first squeezing excess disinfectant into a sink to prevent splash to face, thoroughly wipe the interior, followed by the foam band, strap exterior, and exterior of the visor.
- 3. Ensure all surfaces remain wet with disinfectant for at least 1 minute (or applicable disinfectant wipe contact time).
- 4. Equipment may be rinsed with tap water if visibility is compromised by residual disinfectant.
- 5. Allow to dry (air dry or use clean absorbent towel).
- 6. Remove gloves and perform hand hygiene.

April 12, 2020





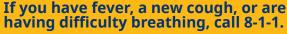

Non-medical inquiries 1-888-COVID19 (1888-268-4319) (ex. travel, physical distancing): or text 604-630-0300

